


## Urinalysis with Reflexed Culture Order and Collection Information

### New UA - C&S Collection Options

**Orders: Urinalysis (UA) or (UA + Microscopic)**

PS# 86206      PS# 907463  
10ml YELLOW      Urine Transfer Straw




PS# 86207  
Kit Includes  
UA tube +  
Transfer Straw

---

**Orders - Urine Culture & Sensitivity(C&S)**

PS# 92179      PS# 907463  
C&S 4ml Gray      Urine Transfer Straw



Mix by  
Inverting  
8-10 times

PS# 91721  
Kit Includes  
C&S tube +  
Transfer Straw

**Orders - Urinalysis (UA + Microscopic) + C&S Reflex**  
C&S reflexes ONLY if the UA is Positive  
Gray Top (C&S) Held for 48 hours for add ons

**Clean Catch Collection**



PS# 91900  
Kit Contains UA/C&S  
tubes + Transfer Straw  
+ 2 alcohol pads

**Foley Collection**



PS# 91901  
Kit Contains UA/C&S  
tubes + LL Transfer Device  
+ 2 alcohol pads

Fill Gray top 1<sup>st</sup>



Use Alcohol prep pads to wipe off tops of tubes and access port

Key points are:

- The gray top tube should be sent to Microbiology. Send the yellow top tube to Specimen Control.
- The new collection and transfer devices are for convenience. Gray top C&S tubes must be filled completely to ensure that culture growth is not inhibited. If samples are short, a urine cup or second yellow top tube are acceptable for culture.
- Urinalysis must be done from yellow top tubes. Other urine chemistries, such as toxicology testing and pregnancy testing, have other tube requirements. Consult the Pathology Handbook for specific specimen requirements.
- Because no reflex order has existed previously, it is up to individual providers to request that it be added to their preference lists.