The Max Air System 710 CAPR™/ <u>Disposable Lens Cuff (DLC) System</u> Powered Air Purifying Respirator (PAPR) Just In Time User Instructions – <u>Respiratory Protection</u> Also See MaxAir CAPR Training Video on Emergency Management Home Page on The Point NOTE: Use of this device requires medical clearance to be on file in UEHC (6-3631)			
		Front Comfort Headband Attachment:	Disposable DLC Lens - (PS 94212 M/L)
		 To attach a front comfort headband strip: align strip parallel to the headband (Velcro only sticks on one side of the strip) and press it on. Flip over and try again if it doesn't stick. 	 2a) Remove protective film by pulling white tab on the outside of the lens. Attach center hole of the lens to the center mounting post on the helmet. Attach the lens to the right and left side mounting posts. 2b) Inside the helmet, pull plastic temple flapper away from lens to rest on sponge material.
CAPR Donning Steps:			
3. Insert power cord to battery pack until click is heard. The unit should power up and self-test immediately. Take note of the LED lights inside front of helmet. Assure at least one green light is lit. Place pack in back pocket, or clip to waist.	 Adjust black fan speed switch located under side rear of helmet to your liking: 1-2-3 equals Low-Med-High air flow. **Most prefer setting "2", however contact lenses users may prefer setting "1". 		
Power cord jack			
5. Loosen the rotary adjustment knob counterclockwise so helmet will easily fit over the head.	 Hold the helmet in one hand <u>while pulling down the cuff</u> <u>under the chin</u> and place helmet on head. To assure good seal, slide fingers up and down between face and cuff. Facial hair and glasses should not interfere with proper functionality. 		
A DIVERSION OF			



Last Updated Feb. 23, 2018