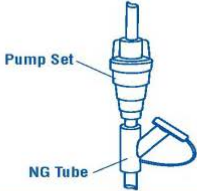

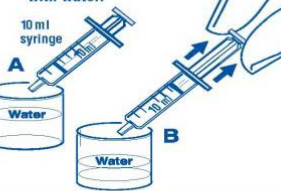
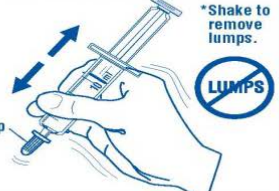

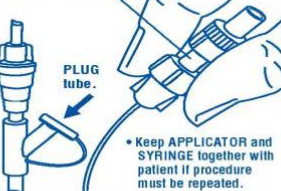


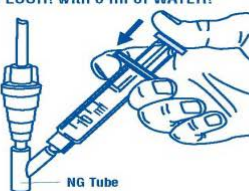
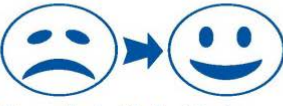



# Clog Zapper<sup>®</sup>

## Instructions for Unclogging NG, G-Tubes and J-Tubes

- You must have a provider order to use Clog Zapper<sup>®</sup>.
- Protective eyewear is strongly suggested. Clog Zapper<sup>®</sup> can cause irritation to the eyes.
- Clog Zapper<sup>®</sup>, once reconstituted, should be used within 24 hours.
  - Refrigerate mixture while not in use.
- Beverages such as Coca-Cola, 7-Up or cranberry juice are acidic and may precipitate the caseinate in formulas. They are not recommended as flush fluids.

<p><b>1.</b> Turn feeding pump OFF and close roller clamp on administration set.</p> 	<p><b>2.</b> Insert CLOG ZAPPER applicator tubing into feeding tube.</p> <p>NOTE: If it is not possible to place the applicator all the way into the feeding tube, the distal end of the applicator may be pressing against the clog. If this is the case, take out the applicator and trim the applicator tubing to the appropriate length. Do not try to break up the clog with the distal end of the applicator.</p> 	<p><b>3.</b> Fill the CLOG ZAPPER syringe to 10 ml mark with water.</p> 
<p><b>4. SHAKE! SHAKE! SHAKE!</b></p> <p>*Shake to remove lumps.</p> <p><b>LUMPS</b></p> 	<p><b>5.</b> Gently fill tube with recommended dosage of 2-5 ml of CLOG ZAPPER solution.</p> 	<p><b>6.</b> Remove applicator from tube.</p> <p>* Keep APPLICATOR and SYRINGE together with patient if procedure must be repeated.</p> 
<p><b>7. KEEP SOLUTION IN TUBING FOR 60 MINUTES</b></p>  <p><b>30 MINUTES MINIMUM</b></p>	<p><b>8.</b> Fill oral syringe with 6 ml of water.</p> 	<p><b>9. FLUSH! with 6 ml of WATER!</b></p> 
<p><b>10.</b> If tubing cannot be flushed, REPEAT steps #4. thru #8.</p>  <p>Always check with physician to determine how many times the process can be repeated.</p>	<p><b>11.</b> After tube clears ...</p> <ul style="list-style-type: none"> <li>Dispose per institution's protocol</li> <li>and RESUME FEEDING.</li> </ul> 	<p><b>CAUTION</b></p> <p>Protective eyewear is strongly suggested. CLOG ZAPPER can cause irritation if it gets in contact with the eyes.</p> <p><b>STORAGE REQUIREMENTS</b></p> <ul style="list-style-type: none"> <li>CLOG ZAPPER, once reconstituted, should be used within 24 HOURS.</li> <li>REFRIGERATE mixture while not in use.</li> </ul>

**INSTRUCTIONS FOR G-TUBE USE**

- Cut APPLICATOR at dot.
- Discard excess tubing.
- Place APPLICATOR into G-TUBE's Administration Set port.
- Continue with NG TUBE INSTRUCTIONS above.

**INSTRUCTIONS FOR LOW PROFILE DEVICES**

- Place FEEDING SET into LOW PROFILE DEVICE following manufacturer's instructions.
- Place APPLICATOR into FEEDING SET.
- Continue with NG TUBE INSTRUCTIONS above.